AxoGuard[®] Nerve Protector
 Distributed by:
 AxoGen, Incorporated
 13631 Progress Blvd., Suite 400
 Alachua, Florida 32615
 Phone: (386) 462-6880
 Toll Free: (888) 296-4361
 Fax: (386) 462-6881
 CustomerCare@AxoGenInc.com
 www.AxoGenInc.com

FP0067-038

CE 0123

EC REP
 Cook Biotech Incorporated
 1425 Innovation Place
 West Lafayette, Indiana 47906 U.S.A.
 Phone: (317) 997-5121
 Toll Free: (888) 299-4224
 Fax: (765) 497-2361

EC Representative:
 Cook Ireland
 O'Halloran Road
 National Technological Park
 Limerick, IRELAND
 Phone: +353 61 334440

©2015 Tradex, Inc. AxoGuard[®] Nerve Protector and its logo are registered trademarks of AxoGen, Inc. U.S. Patents: 6,206,931; 7,652,077; 6,241,981; 6,358,284

Description

The AxoGuard[®] Nerve Protector is a surgical implant that provides non-constricting protection for peripheral nerves. AxoGuard[®] Nerve Protector is designed to be an interface between the nerve and the surrounding tissue. AxoGuard[®] Nerve Protector is comprised of an extracellular matrix (ECM) and is fully remodeled during the healing process. When hydrated, AxoGuard[®] Nerve Protector is easy to handle, soft, pliable, nonflammable, and porous. AxoGuard[®] Nerve Protector is flexible to accommodate movement of the joint and associated tendons, and has sufficient mechanical strength to hold sutures. AxoGuard[®] Nerve Protector is provided sterile, for single use only, and in a variety of sizes to meet the surgeon's needs.

Indications for Use
 AxoGuard[®] Nerve Protector is indicated for the repair of peripheral nerve injuries where there is no gap. The AxoGuard[®] Nerve Protector is supplied sterile and is intended for single use.

Rx ONLY This symbol means the following:
CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
 This product is intended for use by trained medical professionals.

Contraindications

The AxoGuard[®] Nerve Protector is derived from a porcine source and should not be used for patients with known sensitivity to porcine material.

NOTE: This device is not intended for use in vascular applications.

Precautions

- This device is designed for single use only. Attempts to re-process, sterilize, and/or reuse may lead to device failure and/or transmission of disease.
- Do not resterilize device.
- Discard all open and unused portions of the device.
- Device is sterile provided the package is dry, unopened and undamaged. Do not use the device if the package seal is damaged or open.
- Discard device if mishandling has caused possible damage or contamination, or if the device is past its expiration date.
- Do not reuse device prior to rehydration.

Possible Complications

Possible complications can occur with any nerve repair surgical procedure including pain, infection, decreased or increased nerve sensitivity and complications associated with use of anesthesia.

If any of the following conditions occur and cannot be resolved, careful removal of the device should be considered:

- Infection
- Allergic reaction
- Acute or chronic inflammation (initial application of surgical graft material may be associated with transient, mild, localized inflammation)

Storage

The AxoGuard[®] Nerve Protector should be stored in a clean, dry location at room temperature.

Sterilization

This device has been sterilized with ethylene oxide.

Suggested Instructions for Use
NOTE: Always handle AxoGuard[®] Nerve Protector using aseptic technique. Minimize contact with latex gloves.

- Follow standard operating procedures for exposure and mobilization of the nerve. Determine the nerve diameter in millimeters (mm) using a suitable measuring instrument. If necessary, repair the nerve using standard operating procedures. Select an AxoGuard[®] Nerve Protector of sufficient diameter to account for normal edema following traumatic nerve injury. The AxoGuard[®] Nerve Protector diameter should be at least 1-2mm larger than the measured nerve diameter and long enough to cover the affected area.
- Open the outer carton and remove the sterile pouch. Using standard aseptic technique, open the pouch and pass the inner tray to the sterile field for further handling.
- Open the tray and, if necessary, trim the AxoGuard[®] Nerve Protector to the appropriate dimensions for covering the damaged portion of the nerve. The AxoGuard[®] Nerve Protector may be trimmed after hydration, if desired.
- Fill the pre-molded rehydration reservoir with room temperature sterile saline or sterile Lactated Ringer's solution. Hydrate the AxoGuard[®] Nerve Protector for 10 seconds or until the desired handling characteristics are achieved, but not more than 20 minutes.
- Position the AxoGuard[®] Nerve Protector around the nerve (Figure 1). If retracted, re-approximate the nerve ends (Figure 2). If desired, gently flush the device with sterile saline or Lactated Ringer's solution to improve conformability with the nerve. Secure the device as necessary. This may include placing running sutures along the longitudinal slit to enclose the nerve, and/or stay sutures to the nerve epineurium away from the coaptation site. See Figure 3 for completed repair.

Repair of peripheral nerve injuries where there is no gap

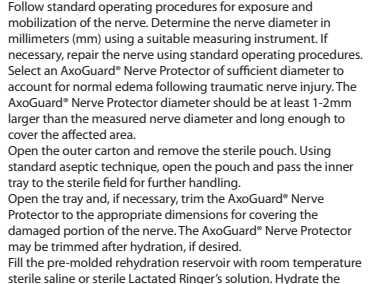
Figure 1

OR

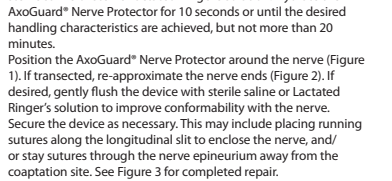
Figure 2

Figure 3

Re-approximate nerve ends

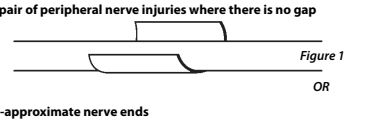
Figure 1

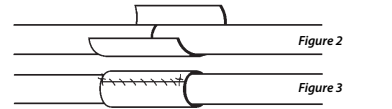
Figure 2

Figure 3

6. Discard any unused portions of the AxoGuard[®] Nerve Protector according to institutional guidelines for biological waste. Do not resterilize.

How Supplied

AxoGuard[®] Nerve Protector is provided in a plastic tray and outer sterile pouch. The pouch is heat-sealed to provide a sterile barrier and has a peelable seal. Contents of the package are guaranteed sterile unless the package is opened or damaged. The AxoGuard[®] Nerve Protector and packaging do not contain natural rubber latex. Do not use AxoGuard[®] Nerve Protector if the peel pouch appears to be open or damaged.

Inquiries

For additional information, to place an order or to report adverse events, contact:
AxoGen Customer Care: 888-AXOGEN1 (888-296-4361)
E-mail: CustomerCare@AxoGenInc.com

Returned Goods Policy

Authorization from AxoGen Customer Care must be obtained prior to returning device. Sterile product must be returned in unopened, undamaged cartons, packed to prevent damage.

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Possible complications

Possible complications can occur with any surgical procedure including pain, infection, decreased or increased nerve sensitivity and complications associated with use of anesthesia.

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